PULPDENT®

Dry-Rite™

Drying Agent

CONTENTS:

- 4 1.2ml syringes drying agent
- 8 dropper tips
- Instructions for use
- MSDS

Caution: Federal law restricts this device to sale by or on the order of a dentist.

For complete drying of dental surfaces

MADE IN USA

Store in cool, dry place

PULPDENT Corporation • 80 Oakland Street • Watertown, MA 02471-0780 • U.S.A.
www.pulpdent.com • Email: pulpdent@pulpdent.com

PULPDENT™

Dry-Rite™
Drying Agent

PULPDENT® Dry-Rite Drying Agent

Contains Ethyl Alcohol

For use in appropriate clinical situations where residual moisture may effect bonding or cementation.

PULPDENT DRY-RITE is a preparation of anhydrous ethyl alcohol with an organic solvent. It is packaged in 1.2 ml syringes for easy handling, and pre-bent needles are provided for controlled dispensing.

INDICATIONS

For use in clinical applications requiring totally dry surfaces. Use Dry-Rite prior to applying hydrophobic pit and fissure sealant or other materials adversely effected by the presence of moisture. It is also used to remove moisture from pits and fissures prior to acid etching.

PRECAUTIONS

1. Dry-Rite with ethyl alcohol is flammable. Handle with care and keep away from open flame.
2. Do not place Dry-Rite directly on dentin as this can desiccate the dentinal tubules.

DIRECTIONS FOR USE

1. Remove syringe cap and place a red dropper tip on the syringe.
2. Isolate the operative area in the mouth to maintain a clean dry field.
3. Rinse the operative area and air dry with clean, uncontaminated, oil-free and moisture-free compressed air.
4. Dispense a small amount of Dry-Rite on the desired surface. For maximum control of flow, do not push on plunger with thumb. It is recommended instead to hold syringe barrel between forefinger and thumb while resting the plunger in palm of hand. Gently pull back with thumb and forefinger to start flow. Use a new dropper tip for each patient.
5. To promote rapid drying, use a gentle flow of clean, uncontaminated air if desired.

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The information presented herein is believed to be factual as it has been derived from the works of persons believed to be qualified experts. However, nothing contained in this information is to be taken as a warranty or representation for which Pulpdent Corporation bears legal responsibility. The user should review any recommendations in the specific context of the intended use to determine whether they are appropriate.

XK-DRY-IN-01

PULPDENT Corporation

80 Oaklant Street • Watertown, MA 02471-0780 • USA
Tel. (617) 926-6666 / (800) 343-4342 / Fax (617) 926-6262
pulpdent@pulpdent.com / www.pulpdent.com