Instructions For Use

PULPDENT® Multi-Cal™

- Non-setting • pH > 12 • Radiopaque

Pulpdent Multi-Cal is a smooth, creamy, non-setting calcium hydroxide preparation in an aqueous hydroxyethylcellulose base with barium sulfate. It is recommended for all clinical applications where calcium hydroxide paste is indicated. It is a dressing, not a hard base, and it provides all the same clinical results as Pulpdent Paste and TempCanal Enhanced.

Indications: Direct Pulp Capping, Pulpal Curettage, Pulpotomy, Dentin Bridge Formation, Temporary and Intermediate Canal Treatment, Periapical Cysts, Abscesses and Lesions, Root Resorption and Apexification

Note: Use a new applicator tip for each patient. To extend shelf life of Multi-Cal, place a drop of water in the cap and recap syringe promptly after each use.

Note: Apply disposable barrier sleeves/wraps over multiple-use dental dispensers before use with each patient. For additional information, refer to: <http://www.fda.gov/MedicalDevices/ProductsandMedicalProcedures/DentalProducts/ucm404472.htm>

VITAL PULP TREATMENT

Multi-Cal is applied directly on the pulp in all pulp capping, pulpotomy and pulpal curettage procedures. It stimulates the pulp to form a new dentin bridge which is visible clinically after 2 - 3 months.

Directions for Direct Pulp Capping:

1. Place a 22-gauge applicator tip on the Multi-Cal syringe and apply a very thin film of Multi-Cal on all exposed dentin and dry with a slow flow of air.

2. Place a bead of Multi-Cal 3mm thick directly over the exposed pulp and dry with a slow flow of air until a crust appears. Multi-Cal does not set hard.
3. Gently tamp the paste with a pellet of cotton to make sure it is in contact with the pulp.
4. Place a thin mix of hard setting cement over Multi-Cal.
5. After cement hardens, place final restoration.

Note: For vital pulpotomy and pulpal curettage procedures, pack pulp gently with sterile cotton until blood clots, then proceed as above. A local anesthetic may be used.

ROOT CANAL TREATMENT


Use routinely as an intracanal dressing between office visits and as a dressing for cysts, abscesses, lesions, etc. Also use for apexification and other hard tissue formation.

Directions for Root Canal Therapy:

1. Prepare and cleanse canal in conventional manner.
2. Place a 22-gauge applicator tip on the Multi-Cal syringe and fill canal with Multi-Cal.
3. Seal access opening with your preferred cement.
4. Multi-Cal is easily removed with file and irrigation.
5. In cases requiring extended root canal therapy, change dressing every month.

Precautions: May be irritating to eyes, skin, respiratory system, gastrointestinal tract or mucous membranes, especially in large quantities and/or with prolonged exposure. For use by dental professionals only. Wear gloves, safety glasses; wash hands after use. Avoid unnecessary exposure.

CLINICAL SITUATIONS WHERE CALCIUM HYDROXIDE CAN BE USED

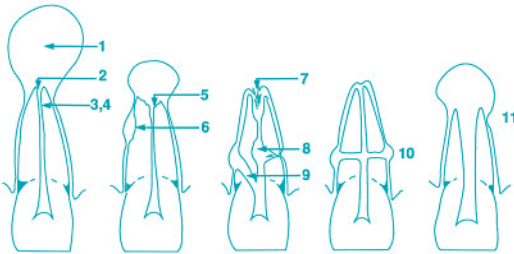1. exudation control
2. large periapical lesions
3. antibacterial dressing
4. temporary root filling
5. apical inflammatory resorption
6. inflammatory resorption following trauma
7. apical internal resorption
8. internal-external root resorption
9. perforations
10. transverse root fractures
11. incompletely developed pulpless teeth

(Heithersay, G.S. Calcium Hydroxide in the Treatment of Pulpless Teeth with Associated Pathology, J. Brit Endo Society, 8[2]: 74-93 July 1975)

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PULPDENT Corporation
80 Oakland Street • Watertown, MA 02472 • U.S.A.