PULPDENT®

Porcelain Etch Gel

9.6% Hydrofluoric Acid

- Thixotropic

CONTENTS:

4 1.2ml syringes etching gel

8 dispenser needles

Instructions for use

MSDS

Caution: Federal law restricts this device to sale by or on the order of a dentist.

For acid etching porcelain veneers and inlays and fractured porcelain surface.
Store in cool, dry place

MADE IN USA

PULPDENT®

Porcelain Etch Gel
9.6% Hydrofluoric Acid

PULPDENT®

Porcelain Etch Gel

9.6% Hydrofluoric Acid *Does not stain resins or resin cements.*

DIRECTIONS FOR USE

1. Protective eyewear must be worn by practitioner, assistant and patient. Protective gloves must be worn by practitioner and assistant.
2. Place rubber dam to isolate operative site and protect soft tissue.
3. Assistant must stand ready at chairside and hold vacuum tip 2-3 inches from porcelain surface during the etching process to evacuate acid fumes.
4. Remove syringe cap from Porcelain Etch Gel and place a pre-bent dispenser needle on the syringe. Carefully start the flow of Porcelain Etch Gel on a pad and immediately dispose of this sample. Do not start flow in patient's mouth. For maximum control of flow, do not push on plunger with thumb. It is recommended instead to hold syringe barrel between thumb and forefinger while resting plunger in palm of hand. Gently pull back with thumb and forefinger to start flow. Use a new dispenser needle for each patient.
5. Carefully apply Porcelain Etch Gel to porcelain surface and leave for one minute while evacuating fumes with vacuum. Some ceramic materials are resistant to etching. To achieve a surface suitable for bonding to these materials, air abrasion is recommended. Creating mechanical retention is advantageous. Cap syringe immediately after use.
6. Rinse surface with copious amounts of water.
7. **If using Embrace™ First-Coat™**, leave porcelain surface slightly moist and follow instructions for use for First-Coat or the Embrace™ WetBond™ Restoration and PFM Repair Kit. **If using Pulpdent Silane Bond Enhancer**, dry porcelain surface with clean uncontaminated air, keep this surface dry, and follow instructions for use for silane.
8. Proceed with repairs of fractured porcelain teeth or with cementation of porcelain veneers or inlays.

PULPDENT PORCELAIN ETCH GEL is a 9.6% hydrofluoric acid gel for etching porcelain veneers and inlays or for etching fractured porcelain surfaces prior to cementation or bonding with resins and resin cements. It can also be used to etch hybrid composites.

CAUTION: Porcelain Etch Gel contains 9.6% hydrofluoric acid, a powerful and aggressive chemical that produces irritating fumes and must be handled with extreme care. Ingestion, contact with skin and eyes, and chronic inhalation of fumes are dangerous.

Care must be taken to protect practitioner, assistant and patient. Pulpdent Porcelain Etch Gel has been buffered, gelled and packaged to minimize the dangers of hydrofluoric acid, provided it is used in accordance with these instructions.

For safe intra-oral use, carefully follow the Directions for Use.

WARNING: If Porcelain Etch Gel comes in contact with skin or soft tissue, rinse exposed area with copious amounts of water. Use high-speed evacuation system to remove water.

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PULPDENT Corporation