

PULPDENT

Embrace™ Esthetic Opaquer

20-Second Light Cure

Product Description

Embrace Esthetic Opaquers are available in bleach white, off-white, and pink shades. A thin coat masks out metals and discolored tooth surfaces for improved esthetics. The material light cures in 20 seconds.

Embrace Opaquers behave favorably in the moist oral environment and form chemical bonds to metal, ceramic and tooth surfaces. The material spreads easily with a brush for fast application.

Note: Apply disposable barrier sleeves/wraps over multiple-use dental dispensers before use with each patient. For additional information, refer to: <http://www.fda.gov/MedicalDevices/ProductsandMedicalProcedures/DentalProducts/ucm404472.htm>

Indications

For masking metal and discolored tooth or restorative surfaces

Contraindications

Not for direct placement on the exposed pulp

Instructions for Use

Ceramic and metal surfaces should be dry prior to applying Embrace Opaquer. Leave tooth surfaces slightly moist. Lightly dry and remove surface water with compressed air or a cotton pellet, but do not desiccate the tooth. Tooth surfaces should be shiny or glossy. Overly wet tooth surfaces will result in decreased bond strength.

1. Prepare surfaces as follows:
 - a. Microabrade **metal and amalgam**
 - b. Microabrade **ceramic** or etch with Pulpdent Porcelain Etch Gel for one minute carefully following instructions for use of etch gel.
 - c. It is not necessary to etch **dentin or cut enamel**.
2. If desired, apply Embrace First-Coat, Pulpdent DenTASTIC UNO or your preferred adhesive to the prepared surface. This step is optional.
3. Using a brush or applicator tip, apply a thin coat of Embrace Opaquer into the etched or abraded surface.
4. Light cure. Embrace Opaquer cures with all lights. Curing time for a halogen light with a minimum of 300 mW/cm² is 20 seconds. More powerful lights will cure faster.
5. Proceed with restoration using your preferred materials.

Caution

Uncured resin materials may cause eye or skin irritation on contact. Dental professionals should wear safety glasses and surgical gloves.

Storage and Handling

- Store tightly sealed in original container at cool room temperature. Avoid direct light, extremes of temperature, contamination and sources of ignition.
- Shelf life of unopened product: 2 years from date of manufacture.
- Re-cap immediately after use.



PULPDENT Corporation

80 Oakland Street • Watertown, MA 02472 • U.S.A.
 (617) 926-6666 or (800) 343-4342 • Fax (617) 926-6262
www.pulpdent.com • pulpdent@pulpdent.com

XP-EMOK-IN-05w
 REV: 11/2020