Root Canal Sealer

Directions For Use

ENGLISH

PULPDENT ROOT CANAL SEALER

DESCRIPTION

Pulpdent Root Canal Sealer is a permanent root canal filling material for primary and permanent teeth that meets ANSI/ADA Specification No. 57-2000 for Endodontic Sealing Materials.

Pulpdent Root Canal Sealer is specially manufactured for placement with either a Lentulo paste filler or the Pulpdent Pressure Syringe. The sealer sets in 2 - 3 hours and does not shrink upon setting to ensure a positive seal. Slight overfills often resorb in 10 - 12 months. After setting, the sealer can be drilled for a post, and it can be removed with gates glidden or peeso drills, if necessary. Pulpdent Root Canal Sealer resorbs with the roots of deciduous teeth.

INDICATIONS

Use alone or in conjunction with gutta percha as a root canal sealer in permanent teeth. Use alone as a root canal sealer in primary teeth.

CONTRAINDICATIONS

Not for temporary filling of root canals.

TECHNICAL DATA

Powder ingredients: zinc oxide, calcium phosphate, zinc stearate, and barium sulfate

Liquid ingredients: eugenol and Canada balsam

Standard: ANSI/ADA Specification No. 57-2000 for Endodontic Sealing Materials

Working time: 30 minutes minimum

Setting Time: 2-3 hours

Storage: Store at room temperature (72° F / 22° C). Avoid direct sunlight.

INSTRUCTIONS FOR USE

Measuring Procedure: Fluff powder by inverting bottle. Measure level scoops of powder. Do not pack powder into scoop. Hold sealer liquid bottle vertically downward when dispensing drops.

Instructions for Use with Lentulo Technique: Mix 2 scoops powder with 4 drops liquid, or multiples thereof, and place in canal using a Lentulo paste filler.

Instructions for Use with Pulpdent Pressure Syringe:

Visit: <https://www.pulpdent.com/shop/category/pulpdent-pressure-syringe/> or request at info@pulpdent.com

STORAGE AND HANDLING

• Store tightly sealed in original container at cool room temperature. Avoid direct light, extremes of temperature, contamination, and sources of ignition.

• **Shelf life of unopened product:** 3 years from date of manufacture. Liquid darkens with age, but this has no effect on performance.

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